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TERMS OF USE OF THE VR VITALIS APPLICATION FROM VR LIFE s.r.o.

VR VITALIS Application as the **"Application"**.

Virtual reality glasses as the **"VR glasses"**.

User using the Application as the **"User"**.

VR LIFE s.r.o. as the **"Company"**.

Responsible worker and supervisor of the User as the **"Worker"**.

These terms and conditions are intended especially for Workers

Purpose of use of the Application

- The application serves to motivate users to exercise and activate the locomotor apparatus, or to relax and exercise cognitive functions.
- It is not used for treatment or prevention and does not compensate for the negative health condition of users.

The application is intended exclusively for the following VR Glasses

- VR glasses: OCULUS QUEST 2 (or META QUEST 2)
 - These are VR glasses that do not need to be connected to a computer, so they are wireless and safer to use the Application.
 - The appropriate application can be used to share the image on a mobile phone or tablet.
- Always use VR glasses according to the manufacturer's or supplier's instructions. Read the manual and instructions for use. Always pay attention to safety when using it.
-

Before each use of the Application

- Check if there is an update available for the Application or a new firmware version for the VR glasses and if so, install the update before using the Application.
- Choose a suitable room that has enough free space to use the Application. We recommend a minimum of 2x2 meters. As part of the Walk in the Park module, we recommend at least 4x4 m for walking Users.
- Define the given space according to the manufacturer's instructions, even in the VR glasses, so that there is no collision with walls or objects.
- Note that if the User is wearing VR glasses, they CANNOT see the surrounding area.
- There MUST NOT be any furniture or other obstacles in the area designated for the use of the Application and also defined in the VR glasses. The floor must be level, not slippery.
- Have the User put on the VR glasses according to the manufacturer's instructions, making sure they are properly positioned (lenses in the center of the eyes for image sharpness) and tight.
- If controllers are used with the Application, make sure to put the lanyards on your hands.
- Check the VR glasses for proper operation and damage.

- Carefully instruct the User on how the Application works, how long they will use it and what to do if the following health problems occur:
 - If the User does not feel well, they should immediately inform the present Worker.
 - In case of nausea, close their eyes and possibly take off the VR glasses.
 - Not to move outside the defined area, which is displayed in the VR glasses with a colored "net".
- When using the app for the first time, we recommend using the Application for only 5 minutes and gradually increasing the time.

While using the Application

- Monitor the User all the time, do not leave the room.
- Make sure the User does not collide with objects or walls or other people.
- In case of nausea or other problems, we recommend the User closing their eyes or taking off the VR glasses.
- Not to use the Application for more than 30 minutes at a time.

After using the Application

- After using the Application, we recommend that the User sits or lies down for 15 minutes and observes whether he/she is okay.
- Disinfect and inspect used VR glasses.

Final Provisions

- The application is not certified as a medical device within the meaning of Regulation (EU) 2017/745 of the European Parliament and of the Council of April 5, 2017 on medical devices.
- The Company bears no responsibility for the use of the Application in violation of these terms of use, legal regulations, instructions and any other documentation.

In on

.....

Name and signature of the Worker - confirmation of familiarization and agreement with these terms and conditions